# Alcohol-Free Surface Disinfectant Wipes


For the surface disinfection of medical devices, including but not limited to, dental chairs, examination lights and patient transport.

Specially formulated for use in the medical and dental environment. Not intended for use on medical and dental instruments.

**BPR** For the disinfection of surfaces.

Use biocides safely and sustainably. This product should be used in accordance with the product label. PT2 Formulation type: AL.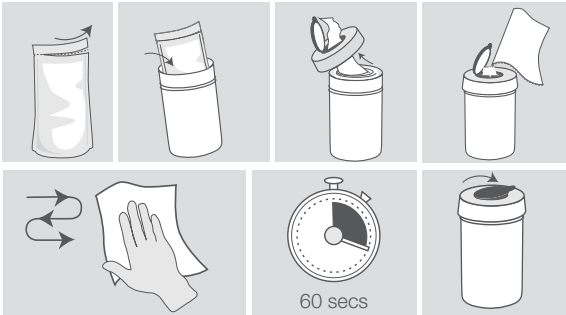

**Directions for use:** 1. Remove cap of dispenser tub, 2. Cut open air tight plastic pouch across the top of the pouch, 3. Feed the centre wipe through the hole in the cap and replace, 4. Pull wipe at a 45° angle. Using sufficient wipes for each application wipe the clean surfaces to be disinfected so that they are completely moistened and let the disinfectant take action over 60 seconds. Allow surface moisture to dry or wipe with clean tissue. Contains detergent. For professional use only.

**Efficacy (60 secs):** Bactericidal. Yeastcidal. Virucidal activity against enveloped viruses: Vaccinia virus, Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

**Tested to Standards:** EN 16777, EN 16615, EN 14885, EN 14476, EN 13727, EN 13624.

**Composition:** Contains Aqua, Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16)) (68424-85-1) 0.48% w/w (4.8g/kg), Didecylidimethylammonium Chloride (7173-51-5) 0.22% w/w (2.2g/kg), Perfumes.

**Warning.** H412: Harmful to aquatic life with long lasting effects. **First Aid procedures.**

P301+P310: IF SWALLOWED: Immediately call a POISON CENTER or doctor. UK number for poison advice: 111. Poisons Information Ireland: For information or to report a poisoning incident contact The National Poisons Information Centre (01 8092166). P305+P351+P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **Handling and Storage.** Ensure there is sufficient ventilation of the area. The wearing of protective gloves is recommended. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Mild steel, aluminium, copper. **Disposal.** P273: Avoid release to the environment. Dispose of contents/container in accordance with local regulations. Do not discharge liquid into drains or rivers. Do not flush. Non maceratable.

## Efficacy

| Microorganism            | Time          |
|--------------------------|---------------|
| <b>Bactericidal</b>      | <b>60 sec</b> |
| Pseudomonas aeruginosa   | 60 sec        |
| Staphylococcus aureus    | 60 sec        |
| Enterococcus hirae       | 60 sec        |
| <b>Yeasticidal</b>       | <b>60 sec</b> |
| Candida albicans         | 60 sec        |
| <b>Enveloped viruses</b> | <b>60 sec</b> |
| Vaccinia virus           | 60 sec        |
| Hepatitis B virus (HBV)  | 60 sec        |
| Hepatitis C virus (HCV)  | 60 sec        |
| Hepatitis Delta virus    | 60 sec        |
| HIV                      | 60 sec        |
| Influenza virus          | 60 sec        |
| Coronavirus              | 60 sec        |
| Herpesviridae            | 60 sec        |
| Filoviridae (Ebola)      | 60 sec        |
| Rubella virus            | 60 sec        |
| Rabies virus             | 60 sec        |
| Measles virus            | 60 sec        |
| Poxviridae               | 60 sec        |

UFI: 2520-0057-K00H-5Q24

Manufactured in the UK by: Topdental (Products) Ltd, 12 Ryefield Way, Silsden, W. Yorks BD20 0EF England, UK. Tel +44 (0)1535 652 750 www.topdental.co.uk

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